


COVID-19 Disinfection for Eye Protection 8.5.20 0930

Spectrum Health contact: Jen Grile *Highlight denotes new content

Seq. No	Task Description:	Key Point / Image / Measure (what good looks like)	Who
<p><i>The following steps should be followed for disinfection goggles, face shields, and surgical eyewear ONLY. Masks with face shields do not apply – these should be disinfected and stored following the mask guidelines.</i></p>			
1.	Obtain eye protection	Goggles or face shield  Face shields are preferred for the care of confirmed or suspected COVID-19 patients	Health Care Worker
2.	Inspect eye protection for signs of deterioration (cracking, clouding, loss of sheen, compromised integrity, etc)	<ul style="list-style-type: none"> If found, discard eye protection 	Health Care Worker
3.	Perform hand hygiene prior to donning eye protection	<ul style="list-style-type: none"> Do not touch the front of the goggles or face shield. 	Health Care Worker
4.	Wear eye protection continuously and for multiple patient encounters	<ul style="list-style-type: none"> Follow extended-use guidelines 	Health Care Worker
5.	Disinfect eye protection immediately upon doffing	<ul style="list-style-type: none"> Don gloves Wipe the eye protection with hospital-approved disinfectant. <ul style="list-style-type: none"> Allow for contact time per label. Wipe off any residue with a paper towel. Remove gloves. Perform hand hygiene. Place goggles or face shield in isolation cart or on wall hook. 	Health Care Worker